

# Instructions (1)

Electronic alarm-system, wireless,  
for children who wet their bed (nocturnal enuresis)

**Pipi-Stop®** **AntiNass®**



## ► INSTALLATION OF THE SYSTEM

### Preparation of the transmitter and the urine-detector



1. Preparation of the elements of the Pipi-Stop

2. Unpack the urine-detector



3. Unpack the transmitter


4. Fix the transmitter on the urine-detector.  
The polarity does not matter.

5. Prepare the paper / towel



6. Place the urine-detector on the paper in the middle diagonally (transmitter above / Pipi-Stop must be legible)

7. Fold the paper on the urine-detector

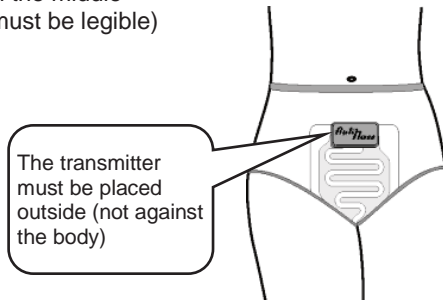

8. If necessary affix the paper to avoid a direct contact of the urine-detector with the skin

9. Place the urine-detector in the slip, on the child's tummy, with the tongue between the legs.  
**Do not fold the urine-detector, but it is normal that it takes the form of the body.**

**Preferably place the urine-detector and the transmitter between 2 pairs of underpants. This will avoid possible false alarms due to the perspiration and will allow fixing better the elements.**

### Commissioning of the receiver



10. Put the receiver on the bedside table (out of children's reach)

11. Insert the round plug of the power supply in the socket at the rear of the receiver

12. Plug the power supply into an electrical outlet 230V → standby is confirmed by a blue light + acoustic signal