# Instructions (2)

Electronic alarm-system, wireless, for children who wet their bed (nocturnal enuresis)

## AFTER THE NOCTURNAL MICTION



Remove and throw the paper in the bin (change the paper every day to avoid false alarms)



Clean the urine-detector and the transmitter under tepid running water







Separate the transmitter and the urine-detector. This stops the alarm! If the alarm was longer than 3 minutes, press briefly one of the two loudness push-buttons on the receiver to stop the alarm





Wipe the urine-detector and transmitter dry. Do not rub the urine-detector!

#### Report the result of the night on the diagram / Do not reinstall the Pipi-Stop for the rest of the night.

### TEST OF THE SYSTEM A test with water does not give an alarm!

#### Test of the receiver



push-buttons

Insert the round plug of the power supply at the rear of the receiver



Shunt out the 2 snap fasteners of the transmitter with a metallic coffee spoon

✓ Sound + white flashing light

If there is no alarm: check that the receiver and the transmitter are not too far away from each other.

► ADJUSTMENT OF THE LOUDNESS (if necessary) Please do not press both push-buttons simultaneously!

#### **Reduce volume:**

press the left button  $(\downarrow)$  and hold until you reach the desired loudness

Plug power supply into an electrical outlet 230V

Press briefly on one of the 2 loudness

✓ blue light is lit + sound



#### Increase volume:

press the right button (1) and hold until you reach the desired loudness

The loudness is automatically stored (even when a power failure occurs).

If the Pipi-Stop system does not work after these checks, please contact us immediately.

## SECURITY / MAINTENANCE



- Receiver and power supply should never be in contact with water (danger of electrical shock!)
- The transmitter should never be opened. The battery can only be changed by Melebi SA (guarantee of water tightness!)
- Use only our 12V power supply with our receiver to avoid overvoltage!
- All the repairing is exclusively done by Melebi SA.

Please return us by mail in the original box after the treatment:

- The completed diagram
- All the rented parts (refer to the picture in the cover of the box

# Test of the whole system

Fix the transmitter on the detector and

(urine-detector must be unpacked)

✓ Sound + white flashing light

shunt out the tongue with a coffee spoon



#### Transmitter at the rear